STOMOCUR® EmplaSectal

Instructions for use EN

Please read the instructions for use carefully, as they contain important information on how to use your STOMOCUR® products.
Intended purpose: <p>STOMOCUR® EmplaSectal is designed for people with an artificial intestinal outlet. The adhesive remover is used to easily remove the skin protection or adhesive surfaces of stoma supplies without irritating the skin.</p>
Applying STOMOCUR® EmplaSectal: <ol style="list-style-type: none">Dampen any adhesive residue left on the skin with EmplaSectal. Remove the residue from the skin. After application, clean the skin with a damp cloth.

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Warnings / Storage conditions EN

- Please only use the STOMOCUR® product indicated in these instructions for its intended purpose (see 'Intended purpose').
- Only use EmplaSectal externally. Do not inhale the vapours and do not allow EmplaSectal to contact open wounds or the mucous membrane. EmplaSectal removes grease from the skin.
- The container is pressurised. Keep away from sunlight and temperatures above 50 °C. Do not open by force or burn even after use (misuse). Do not spray onto flames or burning objects. Keep away from ignition sources – do not smoke. Keep out of the reach of children.
- If you have any questions or problems during use, please contact your specialist doctor or your stoma therapist.
- If a serious injury (incident) occurs in connection with the use of EmplaSectal, please contact your local distributor or the manufacturer.

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