One-piece urostomy system

STOMOCUR® CE

Instructions for use

Please read the instructions for use carefully, as they contain important information on how to use your STOMOCUR® products.

One-piece urostomy system

Intended purpose: Urostomy pouches are designed for people with an artificial bladder outlet. Urostomy pouches are used for the collection of urine and are designed to be single-use. They have a lockable outlet valve through which the collected urine can be emptied.

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URO SAFE, URO SAFE CONVEX, URO SAFE MICROSKIN, URO SAFE MINISTRAP
Urostomiebeutelsystem

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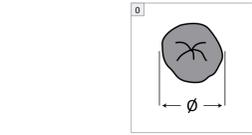
Cut the opening of the skin protection plate to the size of your stoma using a pair of scissors (Fig. 3).

Make sure that you pay attention to the maximum marked cut to fit size of the skin protection plate. To avoid inconvenience, please close the valve with the sealing cap.

Note: Urostomy pouches are additionally extended with a **uro-adaptor** (if required). This provides a further connection to the uro-adaptor and a urine collection pouch (Fig. 3). A "click" can be heard when it is correctly attached.

After connecting the uro-adaptor to the uro-pouch, open the turn valve (180° clockwise) to ensure flow.

Close the soft drain valve of the pouch before attaching it (Fig. 2a). The turn valve is set in the closed position at the factory (Fig. 2b). To avoid inconvenience, please close the valve with the sealing cap.

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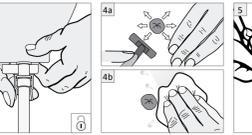
Place the skin protection cut-out on the lower edge of your stoma. Carefully stick this to the skin surrounding the stoma, starting at the belly button (Fig. 6).

Remove the hair (if necessary). Cover the stoma with a compress (disposable). To avoid injuries, carefully move the stoma in the direction away from the stoma (Fig. 6).

Clean the skin around the stoma with water and a disposable wipe (moving away from the stoma – Fig. 6a) and pat the area dry.

Do not use any products which have a pH value of less than 5.5. Do not use any products which have a pH value of more than 9. Do not use any products which have a pH value of 5.5 to 9. Do not use any products which have a pH value of more than 9.

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Bruksanvisning	UK
Read the instructions carefully as they contain important information about the use of your STOMOCUR® products.	
One-piece urostomy system	
Intended use: The urostomy bag is designed for people with an artificial urinary bladder. The urostomy bag is used to collect and absorb urine. It is designed for single use only. It has a lockable outlet which allows you to empty the collected urine.	

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