STOMOCUR® Ostomy Belt

Instructions for use

EN

Please read the instructions for use carefully, as they contain important information on how to use your STOMOCUR® accessories.

Intended purpose:

The ostomy belt is designed for people with an artificial intestinal outlet. It is used to provide an additional fixing point for the STOMOCUR® supply system on the body.

Using the ostomy belt:

1. Adjust the ostomy belt to your waist circumference.
2. Carefully stretch the ostomy belt to connect the hook of the belt with the eyelet on the supply system.
3. Finally, straighten the belt.
4. The ostomy belt should be washed after repeated use.
(Hand wash at 30 degrees)

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Warnings

- Please only use the STOMOCUR® product indicated in these instructions for its intended purpose (see 'Intended purpose').
- STOMOCUR® products must not be disposed of in the WC. Please dispose of them in the correct way.
- If you have any questions or problems during use, please contact your specialist doctor or your stoma therapist.
- If a serious injury (incident) occurs in connection with the use of this medical device, please contact your local distributor or the manufacturer, as well as your competent local authority.
- Contains viscose, rubber-coated elastane and cotton.

Storage conditions:

- Store in a cool, dry place away from light.

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