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## STOMOCUR® SGX - Gelling Sachets

## Instructions for useEN

**Please read the instructions for use carefully, as they contain important information on how to use your STOMOCUR® accessories.**

### Intended purpose:

**STOMOCUR® SGX - Gelling Sachets** are designed for people with an artificial intestinal outlet. **SGX - Gelling Sachets** serve to convert thin fluid excretions into a solid gel to make it easier to drain the pouch.

#### Using STOMOCUR® SGX:

- Add a **SGX - Gelling Sachet** into the pouch when putting it on or after emptying it.
- One sachet will thicken up to 400 ml of fluid excretion, binding it immediately and making it easier to empty the pouch.
- The pouch can be emptied into the toilet as usual.

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## FORLIFEBERLIN

## Warnings

- Please only use the STOMOCUR® product indicated in these instructions for its intended purpose (see "Intended purpose").
- SGX - Gelling Sachets** are not for consumption. The sachets are not toxic, but they should be kept out of reach of children.
- If the contents of the **SGX - Gelling Sachets** get into your eyes, rinse with clean water.
- If you have any questions or problems during use, please contact your specialist doctor or your stoma therapist.
- If a serious injury (incident) occurs in connection with the use of **SGX - Gelling Sachets**, please contact your local distributor or the manufacturer.

#### Storage conditions:

- Store the **STOMOCUR® SGX** in a cool, dry place away from light.

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